



COPY

HemoCue Hb 201+

Summary Flow Chart

Test Preparation

- Pull cuvette out of its locating position
- Press & hold left button until the screen is activated
- 3 Flashes dashed indicates the device is ready to use



Quality Control (QC) Analysis

- QC must be run daily before any patient sample
- Press both buttons together, the QC symbol will show
- Fill cuvette with QC material and insert into holder
- Ensure result is acceptable



Test Procedure

- Clean patient finger, lancet and wipe away the 2-3 drops of blood
- Apply pressure until a further blood drop appears
- Fill cuvette in a continuous process ensuring no air bubbles are present
- Do not overfill cuvette (wipe off excess without drawing blood out of the cuvette)
- Place cuvette into cuvette holder and into reading position on device



Results

- Results are displayed within 15 60 seconds
- Dispose of used cuvette in clinical waste bin



Result Recording

- Record ALL results on HemoCue Result Label (see footnote)
- Remember to include any errors e.g. clotted samples

For full instructions and related documents please refer to: Intranet>Pathology>POCT

Author: S. Street	Checked by: N. Hodges	Approved by N. Hodges	Page 1 of 1
-------------------	-----------------------	-----------------------	-------------

FM/POCT/33/2 Effective Date: 10.07.2019