

Roche Blood Gas Analyser

Capillary Analysis

Summary Flow Chart

Sample Preparation

- Collect sample into capillary tube
- Eliminate air bubbles
- Attach a clot catcher



Sample Analysis

- Log into analyser using your individual username/password
- Introduce capillary tube firmly into the orange fill port (green light appears)
- PRESS aspirate sample and wait until a beep is heard
- Remove capillary but do not discard blood until results obtained



Enter Patient Details

- During analysis, complete all **RED** mandatory fields
- Use Δ and ∇ and SET INPUT VALUE buttons to do this
- Once analysis is complete, a paper report will be printed.



Result Recording

- Transcribe all relevant results in the patient notes & include the following:
 - Test Result(s)
 - Date and Time
 - Location of test
 - Signature
 - Also record any failed attempts
- Log off

**For full instructions and related documents
please refer to: Intranet>Pathology>POCT**