



Roche Blood Gas Analyser Syringe Analysis

Summary Flow Chart

Sample Preparation

- Mix the sample well by rolling in-between hands
- Expel any air and the first few drops of blood
- Attach a clot catcher

Sample Analysis

- Sign into analyser using your individual username/password
- Press syringe firmly into the orange fill port (green light appears)
- Inject sample SLOWLY until a beep is heard
- Remove but do not discard syringe until you have results



Enter Patient Details

- During analysis, complete all RED mandatory fields
- Use Δ and ∇ and SET INPUT VALUE buttons to do this
- Once analysis is complete, a paper report will be printed.



Result Recording

- Transcribe all relevant results in the patient notes & include the following:
 - Test Result(s)
- Date and Time
- Location of test

- Signature
- Also record any failed attempts
- Log off

For full instructions and related documents please refer to: Intranet>Pathology>POCT

Author: N. Hodges Checked by: H Witham Approved by: H Witham Page 1 of 1

FM/POCT/34/3 Effective Date: 02/05/17