

# Cobas b123 Blood Gas Analyser Analysis

## Log into Cobas b123

- Press the blue Log on icon
- Scan your barcode for both username and password fields
- Select “overview” screen if not already on “overview” (top left)



## Pre analysis

- Ensure tests/panels required are selected and ready to use
- Dark Green = Selected
- Light Green = Deselected
- Red = Not Calibrated/Not Ready



## Analysis

- Follow on screen instructions and attach syringe securely to the sampling port
- Sample will be aspirated – Do not inject!
- When prompted, remove and confirm by pressing yes



## Result Recording

- Enter All information required in the mandatory fields highlighted
- Record All results in the patient notes
  - Test Result(s)
  - Date and Time
  - Location of test
  - Signature
- Log off once all measurements are complete

**For full instructions and related documents please refer to: Intranet>Pathology>POCT**